◆ PRODUCT INFORMATION◆

Suction Irrigation Probe

Instruction for Use

The Following information should be read before using this device.

◆ Intended Use:

The AMD Suction Irrigation P is an accessory to the AMD Suction Irrigation systems. The Probes are intended for evacuation and aspiration of fluids during general laparoscopic procedure.

Note:

Single use only, Sterile (Unless package is damaged or opened). All contents are ETO sterilized.

◆ Instruction for Use:

Open in a sterile environment, aseptically assemble the probe onto the suction irrigator. Hold the hub of the probe and screw it onto the front of the handpiece until the two components are fully assembled.

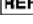
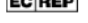
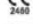
◆ Cautions:

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury.
- Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.
- Do not apply excessive lateral force on the distal end. This may cause the probe crack, break and bend.

◆ Warnings:

- Inspect the device prior to each use for cracks or breaks. Discard the product if there is any damage. Cracks or breaks may lead to inadvertent flow of electricity, shocks or saline leakage.
- Do not use the probe or other laparoscopic instrument as laser backstops. Absorbed radiation may cause excessive heating of laparoscopic instruments. Reflected radiation may cause inadvertent burns.
- The device is suited to AMD Suction Irrigation system only. Attachment of the device to another system may lead to saline leakage, tube block, or system dysfunction.
- Before endoscopic instruments and accessories from different manufacture are utilized together in a procedure, verify compatibility and ensure that electrical isolation of grounding is not compromised.
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."

Explanation of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorised representative in the European community.
	Notified body: DNV Product Assurance AS, with CE2460		
	RX ONLY		

Distributed By:

Fannin UK LTD
Unit 100, Westminster Industrial Estate,
Repton Road, Measham, Swadlincote, DE12 7DT
Tel:01453 874130

Manufactured By:

Advanced Medical Design Co., Ltd.
4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,
24888 New Taipei City, Taiwan

Authorized Address:

OBELIS S.A
Bd. Général Wahis, 53, 1030 Brussels, Belgium
TEL: +3227325954
E-mail: mail@obelis.net

Instruction for Use

Product: Suction Irrigation Probe

The Following information should be read before using this device.

Intended Use:

The AMD Suction Irrigation Probe is an accessory to the AMD Suction Irrigation systems. The probes are intended for evacuation and aspiration of fluids during general laparoscopic procedure.

Note: Single use only, Sterile (Unless package is damaged or opened). All contents are ETO sterilized.

Instruction for Use:

Open in a sterile environment; aseptically assemble the probe onto the suction irrigator. Hold the hub of the probe and screw it onto the front of the hand piece until the two components are fully assembled.

Cautions:

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury.
- Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.
- Do not apply excessive lateral force on the distal end. This may cause the probe crack, break and bend.

Warnings:

- Inspect the device prior to each use for cracks or breaks. Discard the product if there is any damage. Cracks or breaks may lead to inadvertent flow of electricity, shocks or saline leakage.
- Store at 15°C ~25°C.
- Do not use the probe or other laparoscopic instrument as laser backstops. Absorbed radiation may cause excessive heating of laparoscopic instruments. Reflected radiation may cause inadvertent burns.
- The device is suit to AMD Suction Irrigation system only. Assemble the device to other system may lead to saline leakage, tube block, or system dysfunction.
- Before endoscopic instruments and accessories from different manufacture are utilize together in a procedure, verify compatibility and ensure that electrical isolation of grounding is not compromised.
- Dispose used product following local state or federal guidelines.

Caution

Sterilized using Ethylene Oxide

Consult instructions for use

Do not use if package is damaged

Batch code

Do not reuse

Keep away from sunlight

Date of manufacture

Keep dry

Do not re-sterilize

Use by

Authorised representative in the European community

Catalogue number

Manufacturer

Temperature limitation: 15°C ~25°C

Notified body: DNV Product Assurance AS, with CE2460

Distributed by Device Technologies
 1 Garigal Rd, Belrose 2085, NSW, Australia

If further information is required, please contact:

Manufacturer by:

ADVANCED MEDICAL DESIGN CO., LTD.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw

OBELIS S.A
 Bd. Général Wahis, 53, 1030
 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net

◆Instructions for Use◆

Suction Irrigation Set

◆ Indications for Use:

The Suction Irrigation Set is intended for use in minimally invasive abdominal procedures. The set will facilitate irrigation with fluids and suction removal of the waste liquid during the procedure.

◆ Description:

The Suction & Irrigation Set is a sterile, single-use and multifunctional device with which achieves two functions in one, Suction and Irrigation. Available both in 5 mm and 10 mm instrumentation with dual spike tubing.

◆ Cautions/Warnings:

- Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Sterility might be compromised.
- Do not use product after expiration date!
- Do not re-use! If re-used, sterility will be compromised.
- Do not re-sterilize!
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."
- "Federal (USA) and European Union law restricts the device to sale, distribution and use by or on the order of a physician"
- Do not irrigate or aspirate fluids when the active electrode is activated.
- Make sure the back cap is closed firmly when not in use. Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.

◆ INSTRUCTIONS FOR USE

- 1) Remove the sleeve containing the Suction & Irrigation Set from the package.
- 2) Close white clamps on dual spike tubing.
- 3) Remove spike's cap and insert into the fluid bag(s).
- 4) Attach suction adapter to the patient port of the suction unit.
- 5) Open the white clamp to fluid bag once the above steps have been completed.
- 6) Insert the shaft through the desired trocar sheath.
- 7) Operate the irrigation function by depressing the Blue trumpet valve marked "irrigation".
- 8) Operate the suction function by depressing the White trumpet valve marked "suction".
- 9) 2.5mm-3mm instruments can be introduced down the shaft of the suction irrigation device by lifting the back cap. (Ensure the cap is closed when not in use.)
- 10) Upon completion of the endoscopic procedure, dispose of this device in accordance with local regulations.

Glossary of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM-DD
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorised representative in the European community
	Notified body: DNV Product Assurance AS, with CE2460		
	RX ONLY		

Distributed By:

Fannin UK LTD
 Unit 100, Westminster Industrial Estate,
 Repton Road, Measham, Swadlincote, DE12 7DT
 Tel:01453 874130

Manufactured By:

Advanced Medical Design Co., Ltd.
 4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,
 24888 New Taipei City, Taiwan

Authorized Address:

OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 E-mail: mail@obelis.net

◆ PRODUCT INFORMATION◆

Suction Irrigation Set

◆ Indications:

The Suction Irrigation Set is intended for use in minimally invasive abdominal procedures. The set will facilitate irrigation with fluids and suction removal of the waste liquid during the procedure.

◆ Description:

The Suction & Irrigation Set is a sterile, single-use and multifunctional device with which achieves two functions in one, Suction and Irrigation. Available both in 5 mm and 10 mm instrumentation with dual spike tubing.

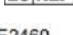
◆ Cautions/Warnings:

- Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Sterility might be compromised.
- Do not use product after expiration date!
- Do not re-use! If re-used, sterility will be compromised.
- Do not re-sterilize!
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."
- "For use by or on the order by a physician, or persons licensed by state law."
- Do not irrigate or aspirate fluids when the active electrode is activated.
- Make sure the back cap is closed firmly when not in use. Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.

◆ INSTRUCTIONS FOR USE

- 1) Remove the sleeve containing the Suction & Irrigation Set from the package.
- 2) Close white clamps on dual spike tubing.
- 3) Remove spike's cap and insert into the fluid bag(s).
- 4) Attach suction adapter to the patient port of the suction unit.
- 5) Open the white clamp to fluid bag once the above steps have been completed.
- 6) Insert the shaft through the desired trocar sheath.
- 7) Operate the irrigation function by depressing the Blue trumpet valve marked "irrigation".
- 8) Operate the suction function by depressing the White trumpet valve marked "suction".
- 9) 2.5mm-3mm instruments can be introduced down the shaft of the suction irrigation device by lifting the back cap. (Ensure the cap is closed when not in use.)
- 10) Upon completion of the endoscopic procedure, dispose of this device in accordance with local regulations.

Explanation of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorized representative in the European community.
	Notified body: DNV Product Assurance AS, with CE2460		
	RX ONLY		

Distributed By:

Fannin UK LTD
Unit 100, Westminster Industrial Estate,
Repton Road, Measham, Swadlincote, DE12 7DT
Tel: 01453 874130

Manufactured By:

Advanced Medical Design Co., Ltd.
4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,
24888 New Taipei City, Taiwan

Authorized Address:

OBELIS S.A
Bd. Général Wahis, 53, 1030 Brussels, Belgium
TEL: +3227325954
E-mail: mail@obelis.net

Instruction for Use

Product: Suction Irrigation Set

The Following information should be read before using this device.

Description:

The Suction & Irrigation Set is a single-use medical device designed for endoscopic surgery for cleansing of the abdominal cavity and aspiration of debris. Suction & Irrigation set is available both in a 5 mm and 10 mm instrumentation with double spike tubing.

Contraindications:

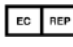
This device is not intended for use when endoscopic techniques are contraindicated.

Preparation:

1. Remove the sleeve containing the Suction & Irrigation set from the package.
2. To remove spike's cap and insert the spike to the desired irrigation pump, please close the white clamp before inserting the irrigation pump.
3. To connect suction adapter (end of white tubing) to patient port of suction bottle machine, please follow instructions provided by suction machine manufacturer.
4. Insert the shaft through the desired 5 mm trocar sheath.
5. The suction and irrigation functions are operated by white and blue trumpet valve. To operate the suction and irrigation feature, please follow instructions provided by suction machine and irrigation pump manufacturer.

Cautions/Warnings:

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
- Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.
- Dispose used product following local state or federal guidelines.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 Caution	 Sterilized using Ethylene Oxide	 Consult instructions for use
 Do not use if package is damaged	 Batch code	 Do not reuse
 Keep away from sunlight	 Date of manufacture	 Keep dry
 Catalogue number	 Manufacturer	 Authorised representative in the European community
 Do not re-sterilize	 Use by	 Temperature limitation: 15°C~25°C
 Medical device	 Single sterile barrier system	 Single sterile barrier system with protective packaging outside
 Notified body: DNV Product Assurance AS, with CE2460	 Distributor	 Importer

If further information is required, please contact:

Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627
 FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
 Email: sales@amdintl.com.tw

OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net
 Jackson Allison Medical & Surgical Ltd.
 56 Lunn Avenue, Mt Wellington, Auckland, New Zealand

Instruction for Use

Product: Suction Irrigation Set

The Following information should be read before using this device.

Description:

The Suction & Irrigation Set is a single-use medical device designed for endoscopic surgery for cleansing of the abdominal cavity and aspiration of debris. Suction & Irrigation set is available both in a 5 mm and 10 mm instrumentation with double spike tubing.

Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

Preparation:

1. Remove the sleeve containing the Suction & Irrigation set from the package.
2. To remove spike's cap and insert the spike to the desired irrigation pump, please close the white clamp before inserting the irrigation pump.
3. To connect suction adapter (end of white tubing) to patient port of suction bottle machine, please follow instructions provided by suction machine manufacturer.
4. Insert the shaft through the desired 5 mm trocar sheath.
5. The suction and irrigation functions are operated by white and blue trumpet valve. To operate the suction and irrigation feature, please follow instructions provided by suction machine and irrigation pump manufacturer.

Cautions/Warnings:

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
- Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.
- Dispose used product following local state or federal guidelines.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

Caution

Sterilized using Ethylene Oxide

Consult instructions for use

Do not use if package is damaged

Batch code

Do not reuse

Keep away from sunlight

Date of manufacture

Keep dry

Catalogue number

Manufacturer

Authorised representative in the European community

Do not re-sterilize

Use by

Temperature limitation:
15°C~25°C

Medical device

Single sterile barrier system

Single sterile barrier system with protective packaging outside

Swiss Authorised Representative

Notified body: DNV Product Assurance AS, with CE2460

If further information is required, please contact:

Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627
 FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
 Email: sales@amdintl.com.tw

OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net

Arazy Group Swiss GmbH
 Bruderholzallee 53, 4059 Basel, Schweiz
 swiss.ar@arazygroup.com

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CE
2460

Cadence®

LAPAROSCOPIC
SUCTION/IRRIGATION DEVICE

INTENDED USE

The Cadence® Suction/Irrigation device is a single use irrigation and aspiration system designed to deliver irrigation fluid into a body cavity and aspirate the fluid and debris during a surgical procedure.

WARNINGS

- This device is for single patient use only and can not be reused or re-sterilised.
- Do not use if package is opened or damaged.
- A complete understanding of techniques and principles involved in electrosurgical procedures is necessary to avoid burn or shock hazard to patient and/or operator.
- Endoscopic procedures must be performed only by surgeons with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.
- Do not operate device without both the shaft cap and secondary cap screwed in securely.
- Do not connect to irrigation bags unless the button clip has been removed.
- Not designed for use with powered irrigation fluid supply systems.

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ACCESSORIES

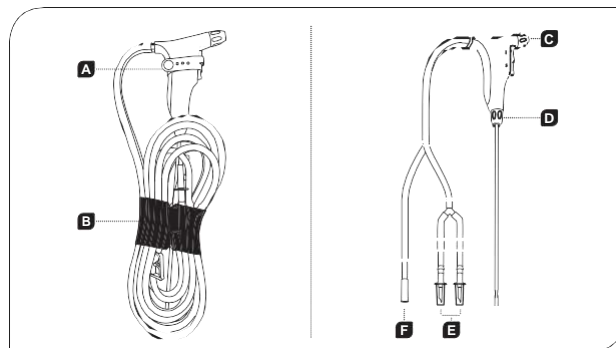
The Cadence® 10mm suction irrigation shaft is designed for use with this handpiece.

USER INSTRUCTIONS

1. The non-sterile nurse peels open the packaging using appropriate aseptic technique and presents the device inside to the sterile instrument nurse.
2. The sterile instrument nurse removes the device and tubing using aseptic technique. The non-sterile nurse can dispose of the packaging.
3. The sterile instrument nurse tears the plastic sleeve surrounding the tubing along the tear line and removes the button clip around the handpiece and discards them.
4. The sterile instrument nurse closes the clamps on the tubing and removes the protective covers from the spikes at the end of the tubing.
5. Using aseptic technique, the spiked ends of the tubing are handed to the non-sterile nurse for insertion into irrigation fluid bag.
6. The non-sterile nurse inserts the spikes into irrigation fluid bag, opens the clamps on the tubing to allow the flow of fluid and attaches the blue suction connector to the suction canister.
7. The sterile instrument nurse depresses the irrigation button to check the flow of irrigation and depresses the suction button to ensure correct operation, prior to use.

ADDITIONAL NOTES

It is recommended the sterile instrument nurse prime and test the device prior to surgeon use. A pressure device may be applied to the irrigation bag when increased flow is required. If shaft is supplied unconnected, the sterile instrument nurse should attach it to the handpiece in the surgeons' preferred configuration, prior to use.

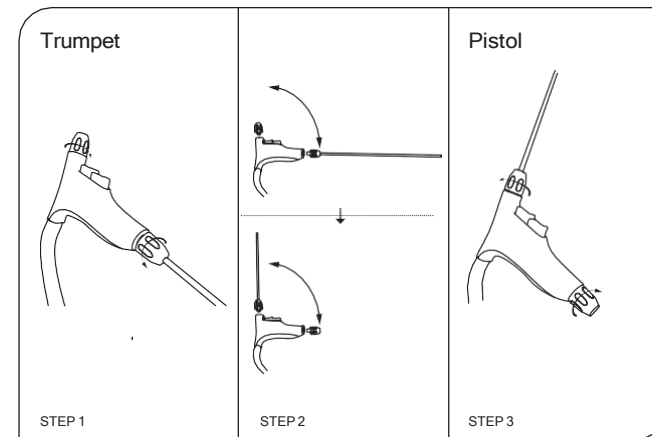
- A** - Button clip, remove before use.
- B** - Plastic sleeve, remove before use.

- C** - Secondary cap.
- D** - Shaft cap.
- E** - Spikes for irrigation fluid.
- F** - Suction connector.

GRIP CUSTOMISATION

The Cadence® Suction/Irrigation device can be used in either trumpet or pistol grip. To swap the grip, unscrew both the shaft cap and the secondary cap, swap positions and reattach both caps securely. Tighten in a clockwise direction.

GOOD DESIGN AWARD®
GOLD WINNER
Received for outstanding design and innovation. Find out more: Multigate.com.au/GDA

Caution	Sterilized using Ethylene Oxide	Consult instructions for use
Do not use if package is damaged	LOT Batch code	Do not expose
Keep away from sunlight	Date of manufacture	Keep dry
REF Catalogue number	Manufacturer	Authorised representative in the European community
Do not re-sterilize	Use by	Temperature limitation: 15°C-25°C
MD Medical device	Single sterile barrier system	Single sterile barrier system with protective packaging outside
CE 2400	Notified body: DNV Product Assurance AS, with CE2400	

Multigate Medical Products
27 Llewellyn Avenue Villawood
NSW 2163 Australia
T +61 (0)2 8718 2888
F +61 (0)2 8718 2889
E enquiries@multigate.com.au
www.multigate.com.au

Advanced Medical Design Co., Ltd.
4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
New Taipei City, 24888 Taiwan R.O.C.
TEL: 886-2-22902627
FAX: 886-2-22988501
https://www.amdintl.com.tw/
Email: sales@amdintl.com.tw

CE MDD
OBTILIS S.A.
88 General Walks, 53, 1000 Brussels, Belgium
TEL: +3227323994
FAX: +3227330081
E-mail: mdd@obtilis.be

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SUCTION AND IRRIGATION KIT
 Reference: AS1200TB
 The following instructions must be read before using the product.

INSTRUCTIONS FOR USE

Indications:
 The suction/irrigation kit is indicated for use in laparoscopy surgery to suction residual fluids and for irrigation during a procedure.

Symbols used:

- See instructions for use
- Sterilized with Ethylene Oxide
- Medical Device
- Oxide Expiry date: YYYY-MM
- Single sterile barrier system
- Date of manufacture: YYYY-MM
- Product
- Manufacturer
- Do not use if packaging is damaged
- Single sterile barrier system with protective packaging outside
- Reference
- Protect from moisture
- See the precautions for use
- Batch N°
- Single use
- Do not re-sterilise
- Approved European Representative in the European Community
- Latex free
- Keep out of sunlight
- Temperature limits: 15°C - 25°C

Contra-indications:

The suction/irrigation kit is not indicated when laparoscopy techniques are contra-indicated.

Instrument description:

The suction/irrigation kit is a disposable, multi-function, single use instrument for irrigation and suction. This kit is available with 5 or 10 mm diameter cannula and with a double perforator.

Instructions for use:

Remove the kit from its packaging.
 Before using the protector from the perforator and inserting it in the bag (or pump), please clamp the tube.
 Before connecting the suctioning part of the tube, please refer to the manual for the suction system.
 Insert the cannula through an appropriately sized trocar.
 Endoscopy and irrigation functions are activated by the white and blue buttons on the "Trumpet" handle. If using a pump, please see the use manual for the machine to optimize pump suction function.
 Use the device in accordance with current regulations.

Precautions:

- This instrument is designed for single use only and must not be re-used or re-sterilized. Re-using or re-handling this device may cause a fatal and result in patient death. Re-using and/or re-sterilizing this device may result in a risk of contamination and infection for the patient. Do not re-use, re-treat or re-sterilize this device. Check the integrity of the individual sterility protection (package) prior to use. Sterility is guaranteed so long as the package has not been opened or damaged.
- Endoskopiske indgreb skal udføres af kirurger som legerer kirurgiske endoskopiske tekniker og har fulgt rigtige oplæring. Skik den meddelte dokumentation om tekniske, komplikationer, kontraindikationer og risici, før der foretages sådanne indgreb.
- Prior to use, check the compatibility of the various instruments used during the procedure.
- Opbevar ved omgivelsestemperatur på 15°C og 25°C.
- Fr.: Set of irrigation/aspiration per gravide/2 percuteurs
- En.: Gravity suction/irrigation set/double spike
- De.: Saug-Schwere-Spülsystem/zwei Dornen
- Es.: Set de irrigación/aspiración por gravídada/obspiga II
- Si.: Sistema di irrigazione/aspirazione a gravida/impostazione a sacca doppia
- Pt.: Conjunto de irrigação/aspiração por gravidade/espigas duplas
- Da.: Suge/skyllesætmetodifærse/døbbelspige
- No.: Gravity/Sug/Spyl set/Toppis
- Sv.: Irrigationsansaug/vdr/kr/ir/aspiration/ububelspets
- Ni.: Spool/zugsysteem/zwaerwactkr/dubbelspike

If you would like further information, please contact:
Distributed/Distribué par:

Peters SURGICAL
 Immeuble AURELIUM - 1 cours de l'Écluse
 92100 BOULOGNE-BILLANCOURT FRANCE
 Advanced Medical Design Co., LTD
 4F-5F No. 29, Wuqian Shi Rd., Wugu Dist.
 24888 New South Wales, Australia
 Tel.: +886-2-9290277 Fax: +886-2-9298801
 E-Mail: info@gmndt.com.tw

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Unternehmensname: OEBLIS S.A