

Instruction for Use

Product: Insufflation Filter with Tubing Set

The Following information should be read before using this device.

Intended Use:





















To be used in conjunction with MIS procedure kit for intra-abdominal Laparoscopic surgery and procedure.
 Note: Single use only, Sterile (Unless package is damaged or opened). All contents are ETO sterilized.

Instruction for Use:

1. Tear off cover to access sterile contents.
2. Place contents on sterile field.
3. Give the connection fitting closest to filter to the nurse for connection to output insufflation gas port.
4. For barbed output, simply cut off fitting and insert over barbed fitting.
5. Connect the opposite Luer connector in sterile field to Veress Needle or gas port of Surgical Trocar Cannula.
6. Proceed with insufflation.

Cautions/Warnings:

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
- Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.
- Dispose used product following local state or federal guidelines.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 Caution	 Sterilized using Ethylene Oxide	 Consult instructions for use
 Do not use if package is damaged	 Batch code	 Do not reuse
 Keep away from sunlight	 Date of manufacture	 Keep dry
 Catalogue number	 Manufacturer	 Authorised representative in the European community
 Do not re-sterilize	 Use by	 Temperature limitation: 15°C~25°C
 Medical device	 Single sterile barrier system	 Single sterile barrier system with protective packaging outside
 Swiss Authorised Representative	 Notified body: DNV Product Assurance AS, with CE2460	

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627
 FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
 Email: sales@amdintl.com.tw



OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003



E-mail: mail@obelis.net
 Arazy Group Swiss GmbH
 Bruderholzallee 53, 4059 Basel, Schweiz
 swiss.ar@arazygroup.com